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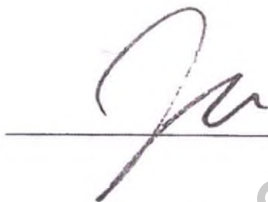
CEO

Dental Studio Co. Ltd.

**INSTRUCTIONS FOR USE
FOR THE MEDICAL DEVICE**

Karl Fritz® medical dental needle holder

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www.roszdravnadzor.gov.ru

1. Device name

Karl Fritz® medical dental needle holder

I. Design versions, 1 pc./pack:

1. Hegar-Baumgartner needle holder, 14 cm, P/N 1001;
2. Crile-Wood needle holder, 15 cm, P/N 1002;
3. Mayo-Hegar needle holder, 16 cm, P/N 1003;
4. Olsen-Hegar EASY needle holder - scissors with inserts, P/N 1005;
5. Mayo-Hegar BLACK needle holder with inserts, 13 cm, P/N 1100B;
6. Hegar-Baumgartner needle holder with inserts, 14 cm, P/N 1101;
7. Crile-Wood needle holder with inserts 15 cm, P/N 1102;
8. Mayo-Hegar BLACK needle holder with inserts, 15 cm, P/N 1102B;
9. Mayo-Hegar needle holder with inserts, 16 cm, P/N 1103;
10. Mayo-Hegar BLACK needle holder with inserts, 16 cm, P/N 1103B;
11. Mayo-Hegar needle holder with inserts, 18 cm, P/N 1104;
12. Castroviejo straight needle holder with inserts, 14 cm, P/N 1105;
13. Castroviejo straight needle holder with inserts, 15 cm, P/N 1105B;
14. Castroviejo curved needle holder with inserts, 14 cm, P/N 1106;
15. Castroviejo curved needle holder with inserts, 15 cm, P/N 1106B;
16. Castroviejo straight needle holder with inserts, P/N 1107;
17. Castroviejo straight needle holder with inserts, 18 cm, P/N 1107B;
18. Castroviejo curved needle holder with inserts, P/N 1108;
19. Castroviejo curved needle holder with inserts, 18 cm, P/N 1108B;
20. Castroviejo curved needle holder, 18 cm, P/N 1109.

II. Instructions for use – 1 pc./pack. (in paper and electronic form)

Hereinafter — instruments, devices, medical device.

2. Information about the manufacturer, developer and authorized representative of the manufacturer

Manufacturer / Developer:

Dental Studio Co., Ltd.

B101, B102, B125, B1, Geumjeong Station 2nd SKV1 Tower, 136 LS-ro, Dongan-gu, Anyang-si, Gyeonggi-do, Republic of Korea

Tel.: +82-31-8069-7007

Fax: +82-31-8069-7009

E-mail: bjkim@dentalstudio.co.kr

Web-site: www.dentalstudio.co.kr

Manufacturer's authorized representative in the Russian Federation

DENTALSTOM LLC

143432, Moscow region, Krasnogorsk, Nakhbino work settlement, Chkalova Str. 7, space LXXV

Tel./fax: +7 (495) 118-76-40

E-mail: info@dn-stom.ru

Web-site: www.dn-stom.ru

3. Intended use of the device/Indications.

The needle holder is designed to hold and guide surgical needles through tissues when suturing during dental surgeries.

4. Potential consumers.

Dental surgeons, maxillofacial surgeons and implantology surgeons.

5. Scope of application, contraindications and possible side effects

Scope of application: dentistry, maxillofacial surgery, dental surgery.

Contraindications: possible allergic reaction to materials.

Side effects: Not known.

6. Precautions.

1. The instruments are supplied in a non-sterile package. It is necessary to sterilize the instruments before using them.
2. Check the instruments for mechanical damage. Before using each instrument, the specialist must make sure that the instruments are in working condition; their surfaces shall be free of any contaminations.
3. The instruments must be used in accordance with their intended purpose and instructions for use.

4. This device can be used only by dentists who received qualifications in the application of appropriate treatment methods and completed training courses.

5. Surgical interventions should be carried out in sterile conditions, medical staff should use masks, gloves, sterile clothes.

7. Description, design of devices and application procedure.

The needle holder is designed to hold and guide surgical needles through tissues when suturing during dental surgeries. These are reusable products.

Product name	Description	Appearance
Hegar-Baumgartner needle holder, 14 cm, P/N 1001;	The jaws are straight. Sizes of needles used: 1-0, 2-0, 3-0, 4-0.	
Crile-Wood needle holder, 15 cm, P/N 1002;	The jaws are straight. Sizes of needles used: 3-0, 4-0, 5-0, 6-0.	
Mayo-Hegar needle holder, 16 cm, P/N 1003;	The jaws are straight. Sizes of needles used: 1-0, 2-0, 3-0, 4-0.	
Olsen-Hegar EASY needle holder - scissors with inserts, P/N 1005;	The jaws are straight. Sizes of needles used: 3-0, 4-0, 5-0, 6-0.	
Mayo-Hegar BLACK needle holder with inserts, 13 cm, P/N 1100B;	The jaws are straight. Sizes of needles used: 3-0, 4-0, 5-0, 6-0.	
Hegar-Baumgartner needle holder with inserts, 14 cm, P/N 1101;	The jaws are straight. Sizes of needles used: 1-0, 2-0, 3-0, 4-0.	
Crile-Wood needle holder with inserts 15 cm, P/N 1102;	The jaws are straight. Sizes of needles used: 3-0, 4-0, 5-0, 6-0.	
Mayo-Hegar BLACK needle holder with inserts, 15 cm, P/N 1102B;	The jaws are straight, with tungsten carbide inserts. Sizes of needles used: 4-0, 5-0, 6-0.	
Mayo-Hegar needle holder with inserts, 16 cm, P/N 1103;	The jaws are straight, with tungsten carbide inserts. Sizes of needles used: 1-0, 2-0, 3-0, 4-0.	
Mayo-Hegar BLACK needle holder with inserts, 16 cm, P/N 1103B;	The jaws are straight, with tungsten carbide inserts. Sizes of needles used: 4-0, 5-0, 6-0.	
Mayo-Hegar needle holder with inserts, 18 cm, P/N 1104;	The jaws are straight, with tungsten carbide inserts. Sizes of needles used: 1-0, 2-0, 3-0, 4-0.	
Castroviejo straight needle holder with inserts, 14 cm, P/N 1105;	The jaws are straight, with tungsten carbide inserts. Sizes of needles used: 5-0, 6-0, 7-0.	
Castroviejo straight needle holder with inserts, 15 cm, P/N 1105B;	The jaws are straight, with tungsten carbide inserts. Sizes of needles used: 5-0, 6-0, 7-0.	
Castroviejo curved needle holder with inserts, 14 cm,	The jaws are curved, with tungsten carbide inserts. Sizes of needles used:	

Product name	Description	Appearance
P/N 1106;	5-0, 6-0, 7-0.	
Castroviejo curved needle holder with inserts, 15 cm, P/N 1106B;	The jaws are curved, with tungsten carbide inserts. Sizes of needles used: 5-0, 6-0, 7-0.	
Castroviejo straight needle holder with inserts, P/N 1107;	The jaws are straight, with tungsten carbide inserts. Sizes of needles used: 5-0, 6-0, 7-0.	
Castroviejo straight needle holder with inserts, 18 cm, P/N 1107B;	The jaws are straight, with tungsten carbide inserts. Sizes of needles used: 5-0, 6-0, 7-0.	
Castroviejo curved needle holder with inserts, P/N 1108;	The jaws are curved, with tungsten carbide inserts. Sizes of needles used: 5-0, 6-0, 7-0.	
Castroviejo curved needle holder with inserts, 18 cm, P/N 1108B;	The jaws are curved, with tungsten carbide inserts. Sizes of needles used: 5-0, 6-0, 7-0.	
Castroviejo curved needle holder, 18 cm, P/N 1109.	The jaws are curved. Sizes of needles used: 5-0, 6-0, 7-0.	

Technical specifications

Product name	Total length, mm	Applied part length, mm	Applied part thickness, mm	Angle, °	Material
Hegar-Baumgartner needle holder, 14 cm, P/N 1001;	140.0	18.0	4.8	No	Stainless steel
Crile-Wood needle holder, 15 cm, P/N 1002;	150.0	18.0	4.8	No	Stainless steel
Mayo-Hegar needle holder, 16 cm, P/N 1003;	160.0	21.0	4.8	No	Stainless steel
Olsen-Hegar EASY needle holder - scissors with inserts, P/N 1005;	165.0	10.0	4.8	No	Stainless steel
Mayo-Hegar BLACK needle holder with inserts, 13 cm, P/N 1100B;	130.0	18.0	4.8	No	Stainless steel
Hegar-Baumgartner needle holder with inserts, 14 cm, P/N 1101;	140.0	18.0	4.8	No	Stainless steel
Crile-Wood needle holder with inserts 15 cm, P/N 1102;	150.0	18.0	4.8	No	Stainless steel
Mayo-Hegar BLACK needle holder with inserts, 15 cm, P/N 1102B;	150.0	18.0	4.8	No	Stainless steel
Mayo-Hegar needle holder with inserts, 16 cm, P/N 1103;	160.0	18.0	4.8	No	Stainless steel
Mayo-Hegar BLACK needle holder with inserts, 16 cm, P/N 1103B;	160.0	18.0	4.8	No	Stainless steel
Mayo-Hegar needle holder with inserts, 18 cm, P/N 1104;	180.0	22.0	4.8	No	Stainless steel
Castroviejo straight needle holder with inserts, 14 cm, P/N 1105;	140.0	13.0	3.5	No	Stainless steel
Castroviejo straight needle holder with inserts, 15 cm, P/N 1105B;	150.0	13.0	5.0	No	Stainless steel
Castroviejo curved needle holder with inserts, 14 cm, P/N 1106;	140.0	13.0	5.0	20	Stainless steel
Castroviejo curved needle holder with inserts, 15 cm, P/N 1106B;	150.0	13.0	5.0	20	Stainless steel
Castroviejo straight needle holder with inserts, P/N 1107;	170.0	13.0	5.0	No	Stainless steel
Castroviejo straight needle holder with inserts, 18 cm, P/N 1107B;	180.0	13.0	5.0	No	Stainless steel
Castroviejo curved needle holder with inserts, P/N 1108;	170.0	13.0	5.0	20	Stainless steel
Castroviejo curved needle holder with inserts, 18 cm, P/N 1108B;	180.0	13.0	5.0	20	Stainless steel
Castroviejo curved needle holder, 18 cm, P/N 1109.	180.0	13.0	5.0	10	Stainless steel

The procedure for using needle holders:

1. Clean, disinfect and sterilize the device;
2. Select an instrument that is suitable in size and purpose;
3. Bring the handles together and clamp the needle with the ends of the needle holder beaks, then apply sutures.

8. Package contents

Karl Fritz® medical dental needle holder

I. Design versions, 1 pc./pack:

1. Hegar-Baumgartner needle holder, 14 cm, P/N 1001;
2. Crile-Wood needle holder, 15 cm, P/N 1002;
3. Mayo-Hegar needle holder, 16 cm, P/N 1003;
4. Olsen-Hegar EASY needle holder - scissors with inserts, P/N 1005;
5. Mayo-Hegar BLACK needle holder with inserts, 13 cm, P/N 1100B;
6. Hegar-Baumgartner needle holder with inserts, 14 cm, P/N 1101;
7. Crile-Wood needle holder with inserts 15 cm, P/N 1102;
8. Mayo-Hegar BLACK needle holder with inserts, 15 cm, P/N 1102B;
9. Mayo-Hegar needle holder with inserts, 16 cm, P/N 1103;
10. Mayo-Hegar BLACK needle holder with inserts, 16 cm, P/N 1103B;
11. Mayo-Hegar needle holder with inserts, 18 cm, P/N 1104;
12. Castroviejo straight needle holder with inserts, 14 cm, P/N 1105;
13. Castroviejo straight needle holder with inserts, 15 cm, P/N 1105B;
14. Castroviejo curved needle holder with inserts, 14 cm, P/N 1106;
15. Castroviejo curved needle holder with inserts, 15 cm, P/N 1106B;
16. Castroviejo straight needle holder with inserts, P/N 1107;
17. Castroviejo straight needle holder with inserts, 18 cm, P/N 1107B;
18. Castroviejo curved needle holder with inserts, P/N 1108;
19. Castroviejo curved needle holder with inserts, 18 cm, P/N 1108B;
20. Castroviejo curved needle holder, 18 cm, P/N 1109.

II. Instructions for use – 1 pc./pack. (in paper and electronic form)

9. Decoding of symbols used in packaging labeling

Symbol	Meaning	Symbol	Meaning
	Catalogue number		Caution
	Lot code		Read instructions for use or see electronic instructions for use
	Date of manufacture		
	Use before		QR-code
	Manufacturer		Non-sterile

10. Conditions of storage, transportation and operation

Storage conditions:

- store in a dry place protected from direct sunlight;
- temperature: +1 to +40 °C;
- relative humidity: 45-55 %;
- atmospheric pressure: 800-1 060 hPa.

Transportation conditions:

Transportation of devices packaged into a transport container shall be carried out by all types of covered vehicles in compliance with the precautions indicated on the transport container and in accordance with the rules of cargo transportation applicable to that type of transport.

- temperature: -50 to +50 °C;
- relative humidity: 0-90 %;
- atmospheric pressure: 800-1 060 hPa.

Indoor conditions during medical device use

- temperature: +1 to +40 °C;
- relative humidity: 45-55 %;
- atmospheric pressure: 800-1 060 hPa.

11. Cleaning, disinfection and sterilization of devices

Used instruments shall be cleaned, disinfected and sterilized before being sent for storage.

1. Pre-sterilization cleaning

Instruments shall be cleaned and disinfected before sterilization.

1) Immerse the instruments with the remaining impurities (blood, soft tissues, bone fragments, etc.) in water. The instruments shall be kept in water before the following cleaning steps are started manually or in an ultrasonic bath.

2-1) Manual cleaning and disinfection of the instruments

a. Immerse the instruments with the remaining impurities in an enzyme disinfectant detergent diluted in water (it is recommended to use products with a neutral pH index or a disinfectant cleaner with similar characteristics (all products must be registered in accordance with the established procedure in the Russian Federation)).

b. Cleaning of all visible contaminations from the surface of surgical instruments is performed using a soft nylon brush with a short pile (cleaning time of each instrument is at least 4 minutes). Cleaning of the delicate applied parts of the instruments is carried out using cleaning needles.

c. The instruments are rinsed under running water to remove any remaining disinfectant.

2-2) Cleaning and disinfection of instruments in an ultrasonic bath (recommended by the manufacturer as the preferred method)

a. The instruments shall be completely immersed in the cleaning solution. It is recommended to use an enzyme disinfectant cleaner (it is recommended to use products with a neutral pH index or a disinfectant cleaner with similar characteristics (all products must be registered in accordance with the established procedure in the Russian Federation)).

b. The instruments are to be cleaned in an ultrasonic bath for 10–20 minutes, then rinsed with water.

c. The instruments are to be rinsed under running water to remove any remaining cleaning disinfectant.

3) Repeat steps a) and b) until the instruments are completely cleaned, then perform visual inspection of the instruments.

4) The instruments are thoroughly dried naturally or using drying cabinets for surgical instruments. If necessary, manual drying using napkins made of non-abrasive lint-free fabric is possible.

5) For more information and data on using the available enzyme and non-enzyme disinfectants, refer to the instructions of the manufacturers of these products.

2. Steam sterilization

After cleaning and disinfecting the instruments, it is necessary to carry out sterilization.

The instruments shall be sterilized by steam under pressure in an autoclave at a temperature from 132 °C to 134 °C for 15 minutes, followed by drying for 30 minutes.

12. Disposal

Medical devices shall be disposed of in accordance with the medical procedures adopted by the medical institutions, and the local, state, or federal regulations. The choice of methods for the safe disposal and destruction of devices depends on the capacity and profile of the medical organization, the availability of waste disinfection/neutralization facilities, the method of neutralization/destruction of waste adopted in the administrative territory (incineration, removal to landfills, disposal).

Instruments are considered epidemiologically hazardous waste (devices contaminated with blood and/or other biological fluids of patients, as well as devices in contact with patients with infectious diseases). In this case, a medical organization only collects and temporarily stores the above-specified waste. Collection is carried out into single-use bags or containers with color labeling adopted for hazardous medical waste in a room or in places for temporary storage of medical waste before subsequent transportation to the place of destruction or disposal. Instruments in contact with the patient shall be pre-cleaned and disinfected in accordance with the established procedure before being disposed of. Transportation of medical waste from medical institutions, and their disposal is carried out by specialized organizations having a license to handle epidemiologically hazardous medical waste.

Consumer package and unused defective devices belong to epidemiologically safe waste, similar in composition to solid household waste (SHW).

13. Shelf life

The shelf life of the device is unlimited.

14. Service life of devices

With proper use, the service life of the instruments is unlimited, except for the effects of natural wear.

15. Maintenance and repair

Maintenance and repairs are not provided.

16. List of international standards that the medical device complies with

EN ISO 13485; EN ISO 14971; EN ISO 14155-1; EN ISO 14155; EN ISO 10993-5; EN ISO 15223-1; EN 1041; MEDDEV 2.7/1 Rev. 4; MEDDEV 2.12/1 Rev. 8; EN ISO 17665-1; EN ISO 11138-1; EN ISO 11737-1; ISO 17665-1; ISO 11135-1; ISO 13397-2.

Note: The latest editions of the standards are used: the organization maintains and updates the list of reference standards as a record of its own management system in accordance with ISO 13485.

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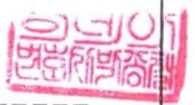
LEE, JUNGYONG -----

attorney-in-fact of
DENTAL STUDIO CO., LTD.
CEO KIM BEOM JUN -----

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appeared before me and admitted
said principal`s subscription to
the attached
INSTRUCTIONS FOR USE FOR

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This is hereby attested on this
20th day of Jan. 2025 at this office.

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This office has been authorized by the
Minister of Justice, the Republic of
Korea, to act as Notary Public Since
7, Feb. 2020 Under Law No.15150.

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